AUGENLIDER

FALTEN

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AUGEN LIFTING Behandlung

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1. EYE MASSAGE

Apply **Age Logic Eye Massage** to the eyelids, under the eyes, the corners of the eyes and the forehead.

Repeat each movement 4 times.

1

Frown lines

Upward fan-like movements between the eyes.

2

Forehead

Alternate horizontal smoothing movements with the flat of the hand.

3

Upper eyelids

Smoothing movements with the thumbs then apply pressure to the temples.

4

Twisting pressure to the left upper eyelids. Repeat to the right upper eyelids.

5

Smoothing movements with the thumbs then apply pressure to the temples.

6

Smoothing movements and pressure with all fingers.

7

Pressure and smoothing movements with all fingers.

Lower eyelids

8

Eye contour

Smoothing butterfly movements.

9

Toning tapping movements.

10

Forehead

Smoothing movements with fingers interlaced then apply pressure to the temples.

2. EYE MUSCLE STIMULATION

10
min

Apply a thick layer of **Age Logic Eye Gel Serum** to the eyelids, under the eyes, the corners of the eyes and the forehead.

Use the electrode balls for all movements.

Repeat each movement 5 times.

Start with the left eye contour.

Adjust the intensity of the stimulation depending on the client's sensitivity (default setting: 5).

1

2


Eye contour

1. Press the electrode onto the skin until it touches the muscle in the corners of the eyes. Move the other electrode back and forth on the eyelid with firm smoothing movements.
2. Repeat this movement under the eyes.

Press the electrode onto the skin until it touches the muscle in the corners of the eyes. Move the other electrode under the eyes, using firm smoothing movements towards the inner corner of the eyes, and circular smoothing movements along the bone in the eye contour, back to the corners of the eyes.

3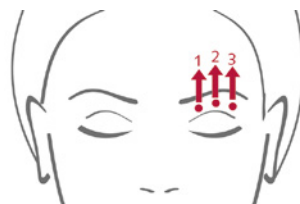

Forehead

1. Press the electrode onto the skin until it touches the brow muscle and move the other electrode to the upper forehead.
- 2 and 3. Repeat this movement towards the outer forehead in three steps.